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ENGLISH

Instructions for Use - AgaChamber® Spacer Extra- HOME / HOSP models

EXTERNAL USE / HUMAN USE - ADULT AND CHILD

PRESENTATIONS:

Adult/Child HOME/HOSP Model	Use from 4 years old
Baby HOME/HOSP Model	Use from 0 to 3 years old
TRAC HOME / HOSP model	Use without age restriction

Package containing 01 Spacer with one mask.

Available colors: Yellow, Blue, Orange, Purple, Pink, Green and Red.

Learn about parts of AgaChamber® Spacer Extra:

Notes: TRAC HOME / HOSP model is formed only by parts 1, 2, 3 and 4, as in tracheostomized patients, medication is administered directly into the trachea.

COMPOSITION

Polypropylene, Elastomers, and Silicone.

INFORMATION TO USER

WHAT IS AgaChamber® Spacer Extra - HOME / HOSP MODEL?

It is a health product, reusable, intended for hospital use, provided the disinfection or sterilization is performed.

WHAT IS THIS PRODUCT FOR?

Assist and optimize the administration of inhaled aerosol medications, available in pressurized dispensers, popularly called "inhaler".

HOW DOES THIS PRODUCT WORK?

It works by directing the spray jet and allowing the retention of large particles (propellant) in order to avoid their deposition in the user's mouth or oropharynx, while simultaneously allowing the smaller particles (active substance) to reach the small gauge airways.

It has a valve (part nº 2) of great importance to ensure the spacer use effectiveness, it ensures a unidirectional flow of the drug (for inspiration), preventing the air exhaled to go into the chamber (barrier against return). This way, there is no dilution of the drug in the next push of spray, ensuring full receiving of the dose.

The product was developed for use with or without a mask, according to the type of patient - children, adults or the elderly - with medical advice.

The mask creates a closed environment, so that the user can not synchronize their inspiration with the timing of the spraying activation, and receives the full dose of the drug, because it will be retained in the mask up until the user breathes. The small mask valve is fixed and should not be removed under any circumstances of use or cleaning, because its function is to allow the air outlet in the patient's exhalation and to avoid the outside air intake.

TO WHAT / WHOM THIS PRODUCT IS INTENDED?

Suitable for patients with respiratory failure (asthma, bronchitis, DPOC, etc.) who are under treatment or care of a health professional.

WHEN SHOULD I NOT USE THIS PRODUCT? / WARNINGS

Keep AgaChamber® Spacer Extra out of reach of children, as it is a product for health and not a toy.

When using on children, it is indispensable the supervision of an adult.

The product should be used only for the purpose it is intended. The product should not have to mask the valve removed even during cleaning.

THE VALVE (PART N° 3) IS ESSENTIAL IN THE MEDICATION ADMINISTRATION TO ENSURE SAFETY AND FUNCTIONALITY OF THE PRODUCT, THUS DO NOT REMOVE IT UNLESS AT THE SANITATION TIME.

PRECAUTIONS

Perform cleaning of the product before the first and after each use, following the recommendations described in this Instructions for Use.

Attention: The discretion of the physician or healthcare provider, as the criticality of the patient's clinical status, the disinfection or sterilization of the product before first use is recommended.

Before using the AgaChamber® Spacer Extra, ensure that there are no foreign objects inside the chamber. If so, repeat the cleaning procedure.

The assembly of the product outside the terms of this operating instructions can cause leakage of the drug with consequent reduction of efficiency in the use to which it is intended.

HOW TO SANITIZE THIS PRODUCT?

Disassemble the AgaChamber® Spacer Extra without removing the valve mask (it should be kept fixed). Wash all parts under **cold running water**, using a few drops of mild liquid detergent, in a soft sponge and water. Rinse.

Let drain the water in excess. Do not rub dry. Allow it to air dry or in material drying oven, with the chamber (part nº 2) in the vertical position.

HOW TO DISINJECT / STERILIZE THIS PRODUCT?

So that the product may have hospital use, it is recommended that in addition to cleaning, disinfection and / or sterilization methods thereof should be carried out, selected according to the medical professional responsible option.

Two methods were tested:

FOR DISINFECTION

Immersion in sodium hypochlorite (in 0.02 and 0.5% concentrations) for 30 minutes.

FOR STERILIZATION

Submission to moist heat under pressure (autoclaving) at 121 °C for 20 minutes.

Notes: The product can be reused for up to 10 cycles of disinfection / sterilization, chamber's opacity may occur from the first cycle, which does not compromise its functionality and efficiency.

If the physician decides to grant the AgaChamber® Spacer Extra for the patient to take home, it is recommended that they instruct how to perform the cleaning of the product.

HOW TO ASSEMBLE THE AgaChamber® Spacer Extra?

The product should only be installed when all parts are completely dry.

Assemble the product in the order shown in Figure illustrated earlier this instruction.

STEP BY STEP:

Attach the base (1) in the chamber (2);
With the chamber vertically, position the side holes of valve (3) in the direction of the fixing pins of the chamber and attach them perfectly. The two valve sides are identical;

After fixation, engage mouthpiece (4) at the same end of the chamber where the valve has been fixed;

If you will not use the Spacer afterwards, snap the mouthpiece cover (5), to protect it from dirt;

If you are to use the Spacer, afterwards, do not fit the cover. There are two options for use: with or without mask. In case of using the mask (6), just fit it directly into the mouthpiece (4).

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